


YM - 9011 Instruction book

Specification

Voltage / Frequency	A C (220-240V/50Hz) (110-120V/60Hz)
watt	300w
Highest temperature	230°C

Name of each part of product

- name of each part of product is shown on picture below :
- Handle
- Temperature Regulator
- Temperature light
- Time light
- Time Regulator

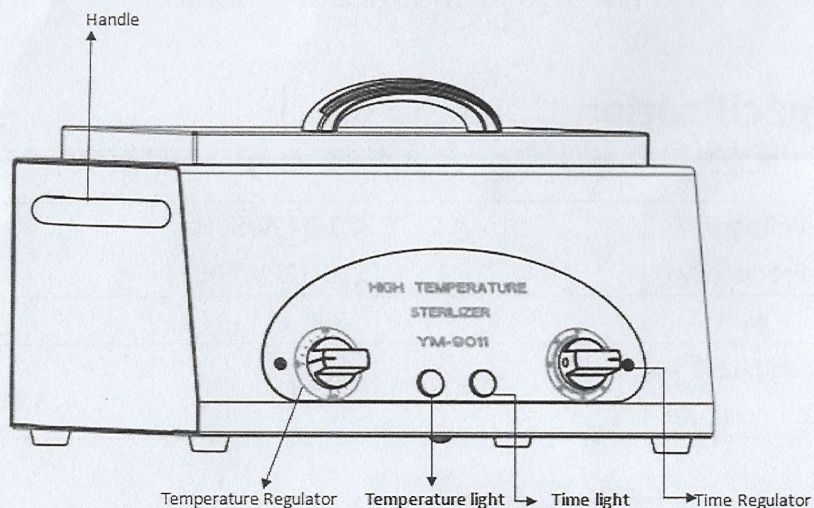

Foreword

- ① Please read the specifications before use this product.
- ② The method of use the products, attention, and maintenance.

Functions

Used to kill the bacteria on tools with high temperature

Using method

Please use the machine in a ventilated environment, the same as after it has been used.

Things should be dry without liquid when put inside to sterilize. If the sterilized things have ingredients of metal, must make sure the metal can bear high temperature.

Power socket at the back of the machine, and fuse box is under the plug jack. Please use something pointed to open the fuse box which is under the plug jack when you want to replace the fuse.

The left knob is for the temperature regulator. If setting up the temperature 100°C, the lamp "1" will be light. When the temperature is over than 100°C the lamp will be turned off, and the lamp will be light again when the temperature returns back.

The right knob is for "ON/OFF" and the time regulator. When the machine is on, the lamp "2" will light.

Maintenance manual

- Clean the product with alcohol cotton regularly, never use industrial solvent or sandpapers cleaning products.
- Disinfection machine should be placed in a dry place.
- Please find professional personnel to help overhaul the machine if encounter some problems.

Safety announcements

◆ meanings of the symbols

-  Danger, Warning, Pay attention to .
 Prohibited
 Cause some problems if not do according the method .

▲ Attention before use

⚠ warning			
	Children can not use this product in case of an accident.		Make sure the quartzites inside tank before use.
	Reading this book before use		Ensure the machine used to correct way, can not be used in out of function place.
	Do not plug on before identifying the voltages of the product and electrical source are congruent.		Different voltage can be caused Fire or Get an electric shock accident.
	Make sure the Power supply with good grounding.		Ensure that disinfection tank is drying before use, Avoiding electrical shock accident.

	Ensure that far away from the damp environment before using. Avoiding cause fire, and electric shock accidents.		Banned used in the bathroom and wet places ,Avoiding fires and electric shock accidents.
	When the hand have water do not use this product, first dry you hands before used again, otherwise may cause electric shock.		When the outside power cord broken and show the inside cord, Never connect the electricity, Avoiding cause Fire or electric shock accident
	After long transportation, avoiding an accident please check the product before using.		Ensure that is stable voltage in electrical before using, prohibited used in a instability of the circumstances.
	Don't transform this product by yourself.		

▲ Attentions when using :

⚠ Dangerous			
	Do not let children touch the products when working.		Ensure that far away from the damp environment before using. Avoiding cause fire, and electric shock accidents.
⚠ Warning			

	Put in a flat desktop when the machine working.		Stop use immediately when occur anomalies reaction.
	if you find the power supply voltage appears instability when the sterilizer at work,stop use immediately, in order to avoid damage the machine or cause accidents.		When the sterilizer appears unusual at work,should stop use and send to the maintenance place, continue to use when the sterilizer is trouble-free.
	When the sterilizer machine is at work, stop use and pull out the power supply immediately if leakage.		When the sterilizer at work, prohibit touching the machine.
	Far away from drunk or un conscious people when the sterilizer at work.		Prohibit to use the machine when no quartzite inside
	Keep quartz sands filling not overmuch, in order to prevent overflow when inserting tools into quartz sands.		Put on gloves and turn off the power switch when take out the tools from tank,be careful of burns
 Attention			
	people should next to it and can not go far away when the machine working, turn off the power switch when finish using. Remember to plug out if don't use of the product in a short time.		

▲ Attentions after use :

 Warning			
	The sterilize force to stop work due to did not work properly, prohibit to use before repair, in cause of accident		Prohibit laypeople to repair the machine when the sterilizer stop working abnormally.
 Attention			
	Pull out the power supply if the disinfection machine don't use for a long time.		The machines should be packaged well if long time not use.
	The sterilizer should put in dry place.		Always far away from children.
	Prohibit use water to clean the machine directly .		Unsuitable to keep the product without use it for a longtime.