


Instructions Manual

Beauty instrument

WKB020

High Tech Air Flow

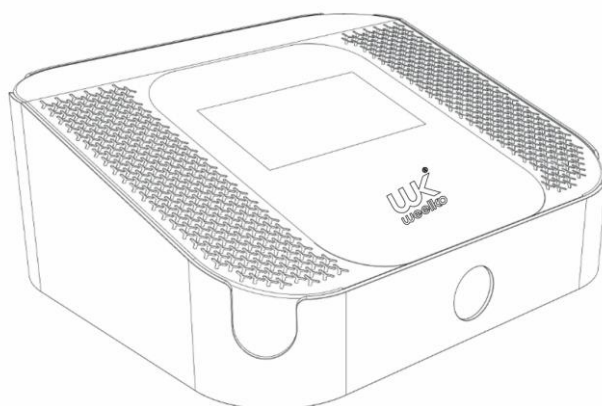

VERSION 1.0

Thank you sincerely for choosing our product. This manual is compiled to help users operate this instrument effectively. We hope our company's products and services will bring you a distinguished experience.

This user manual contains all general information about the instrument. It serves as the starting point for new users to begin using the instrument, as it provides a general introduction to the system, explains how it works, and details the instrument's installation and various operation methods.

Usage Precautions

1. Before turning on the machine, be sure to read the user manual carefully. Operators must be familiar with the user manual and operation methods, and the instrument must be managed by a designated person;
2. When replacing the fuse, always turn off the power switch to prevent fire and electric shock;
3. Must use a power supply of 220V±22V, 50HZ±1HZ and an effectively grounded socket;
4. Using a damaged or modified power cord may cause fire and electric shock hazards;
5. If the instrument is loose, falling, or damaged, please contact the company in a timely manner;
6. Do not use the instrument in unstable environments such as tilting, vibrating, or impacting conditions;
7. No water or debris is allowed to enter the interior of the instrument. In case of foreign objects entering, please contact the company promptly;
8. When moving the instrument, the power plug must be pulled out first;
9. The instrument should be placed on a stable surface, with at least 5cm of space left around it to ensure air circulation and facilitate heat dissipation of the instrument;
10. Do not cover the instrument with anything to prevent blockage of the air vents;
11. Avoid using the instrument in the following environments: direct sunlight; high humidity environments; near water; vibrating or tilting places; strong magnetic field environments; places where electromagnetic waves or impulse voltages occur; places where chemicals are stored or corrosive gases exist;
12. If an abnormal situation occurs to the instrument or the user during operation, the operator shall immediately press the power switch, and stop using the instrument;
13. The instrument shall be correctly used by skilled operators. Personnel other than operators are not allowed to use the instrument;
14. After the operation, clean the suit with towel;
15. After the operation, place the suit to a dry place and better not to bend the tubes;

Contraindicated Groups (Do Not Use If You Belong to the Following Groups)

- Individuals with implanted cardiac pacemakers, defibrillators, or metal implants (such as steel plates, bone nails) in the body;
- Pregnant women, lactating women, and women during menstruation;
- Patients with epilepsy, hypertension, severe heart disease, diabetes, tumors or cancer, or thyroid diseases;
- Individuals with open wounds, skin damage, severe inflammation, eczema, dermatitis on the skin, or those with facial telangiectasia (spider veins) or rosacea;
- Those who have received botulinum toxin injections, filler injections, laser treatments, chemical peels, or plastic surgery recently (usually within 6 months).

Principle and Effects

The pressotherapy, with a set of cloth-like accessory and 32 pieces of air bags (16 pieces of air bags on each side of body), which allows for pressure by cycles or all-together for both the whole body and certain parts of the body, is a compression system designed to increase the venous and lymphatic flow and enhance extra-cellular fluid clearance.

It can help to promote the body's natural toxin clearing functions. The revitalization and oxygenation of the tissue helps to slim and redefine the legs, stomach and arms while enhancing skin tone.

This suit has an advantage, that each air bag (each channel) is half overlapping on the adjacent air bag, making the massage more exquisite and effects better.

And the system supplies 7 different inflating modes for different purpose, and an auto mode A combining this 7 modes.

Mainframe Indication



Suit Width Adjustment




Connect the Suit to Mainframe

There is a hole and a stick to help find the right position. Follow the stick, insert the plug to machine, then rotate the round cover nut, to nearly the bottom (no need 100% bottom)




Display Indication

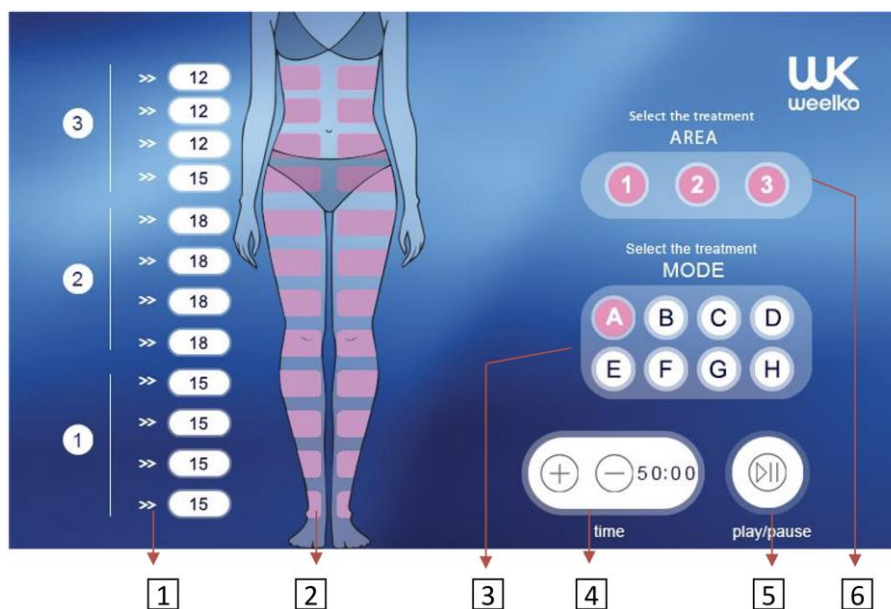

1 Inflating time adjustment for each channel,

Click >> and the duration will cycle from 3s to 21s, with 3s for each step

2 Click the pink area to close the channel directly, for example

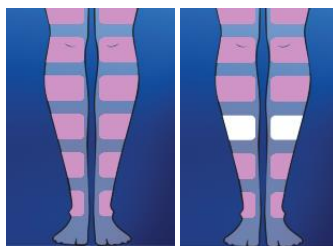

3 Click the letter to change the inflating mode,

Mode A

Combines modes B to H, that is 5 minutes for mode B, then 5 minutes for mode C,,,, then 5 minutes for mode H.

The time for inflating channels cannot be adjusted during operating this mode.

It can be a default choice for daily use.

Mode B

Inflates from channel 1 to channel 12, one channel by one channel, that is inflating 1, then inflating 2, then inflating 3,,,, inflating 12 and repeat.

This mode carries a slight peristaltic massage, mostly used during the maintenance phases or whenever an intensive massage is not necessary.

It is indicated for individuals who do not have circulatory diseases of any kind but want to prevent.

Mode C

Inflates 1, then inflate 1+2, then 2+3, then 3+4, then 4+5,,,, then 11+12 and then repeat.

Lymphatic swelling of a sector while maintaining swelling of the previous one limits lymphatic reflux.

This mode carries an intensive peristaltic massage, which is recommended to people needing an intensive treatment, i.e. featuring an evident presence of cellulitis, fat or general stasis of liquids.

Mode D

Inflates 1, then 1+2, then 1+2+3, then 1+2+3+4, then 2+3+4+5, then 3+4+5+6, then 4+5+6+7,,,, then 9+10+11+12 and repeat.

This mode causes the venous blood to be even more thrust into centripetal sense, thus favoring both the work of the heart and the action of valve containment venous. It is indicated to stimulate and improve blood circulation, by providing more oxygen to the tissues and contributing elimination of harmful substance. It is very effective in the presence of muscle hypotension.

Mode E

Inflates 1, then 1+2, then 1+2+3, then 1+2+3+4, then 1+2+3+4+5, then 1+2+3+4+5+6, then 1+2+3+4+5+6+7,,,, then 1+2+3+4+5+6+7+8+9+10+11+12 and repeat.

This mode is for people with significant water retention and associated with clinical signs of venous insufficiency. And people with lymphatic system disorders also benefit from this mode.

This mode carries an intensive peristaltic pressure massage, and it is concentrated on the abdomen and thighs. This is recommended for clients with obvious cellulite or fat, in which the imperfection extends to the sides of the thighs and abdomen, and is presenting in soft form.

Mode F

Inflates in 2 ways at the same time:

One way is 1+9, 2+10, 3+11, 4+12

One way is 5+9, 6+10, 7+11, 8+12

This mode carries a high frequency massage, popular for whole body's relaxing. Usually used for people without lymphatic problems, or people after sporting.

Mode G

Inflates 1, 1+2, then 3+4, then 5+6, then 7+8, then 9+10, then 11+12 and repeat.

This mode carries a high frequency drainage massage, usually used for light people with lymphatic problem or other sub-health situation.

Mode H

Inflates all together, deflates all together, and then repeat.

This mode is usually used after operating some other modes, to enhance drainage massage quickly for the whole body.

4 Treatment time adjustment

5 Start or Stop adjustment

6 3 shortcuts, corresponding to the 3 divided zones, to switch on or off several channels quickly.



Product Packaging List

Name	Image Quantity
Main Unit	1 pc
High Tech Air Flow Suit	1 pc
Power Cord	1 pc

Product Specifications

- Input Voltage: AC 220V-240V
- Product Frequency: 50/60HZ
- Product Power: 60W
- Pressotherapy Channel: 12 Channels
- Pressotherapy Power: max 250mmHg
- Pressotherapy Mode: 8 Modes
- Packaging Dimensions: 54x41x40CM
- Gross Weight: 14.5KG
- Net Weight: 13.5KG



Imported by
WEEKO BARCELONA SL

Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona - Spain T.
+34 93 824 90 20
E. info@weelko.com